
QyScore Documentation

Release 1.11, rev 1 USA

Qynapse

Dec 02, 2025

1	Legal notices	1
1.1	Copyright notice and trademark acknowledgements	2
1.2	Indications for use	2
1.3	Contraindications	2
1.4	Target patient population	2
1.5	Intended users	2
1.6	Description	3
1.7	Clinical Benefits	3
1.8	Global presentation	4
1.9	Technical description	4
1.9.1	Installation	4
1.9.2	Compatibility with other devices	5
1.9.3	Performances	5
1.9.4	Maintenance	8
1.9.5	Deactivation, Uninstallation and Data Disposal	8
1.9.6	Safety	8
1.10	Required imaging parameters	8
1.11	Normative data	9
1.12	Warnings	10
1.13	Reporting of Serious Incidents	11
1.14	Electronic instructions for use	11
2	Conventions	12
2.1	Mouse	12
2.2	Typography	13
2.3	Symbols	13
2.4	Glossary	13
3	Connect to QyScore	14
3.1	Global presentation	14
3.2	Login	14
3.2.1	Case of first login	14
3.3	Change password	15
3.4	Logout	15
3.5	Forgotten password	15
4	Send data	16
4.1	Global presentation	16
4.2	Via the DICOM server (e.g. PACS)	16
4.3	Web-based data management interface	17
4.3.1	Uploading data	17

4.3.2	Analyses monitoring	18
5	Results summary report	21
5.1	Global presentation	21
5.2	Display the report	23
6	Web-based results visualization interface	24
6.1	Global presentation	24
6.1.1	Login page	24
6.1.2	Visualization page	25
6.1.3	Account management page	26
6.2	Select patient data	27
6.2.1	Choose a patient to load	27
6.2.2	Change the acquisition to display	27
6.3	General visualization	27
6.3.1	Browsing the volumes	27
6.3.2	View imaging modalities	28
6.3.3	View segmented structures	28
6.4	Switch view modes	30
6.4.1	<i>Quantification</i> mode	30
6.4.2	<i>Graphs</i> mode	31
6.4.3	<i>Longitudinal</i> mode	32
6.4.4	<i>3D rendering</i> mode	33
6.5	Change visualization settings	34
6.5.1	<i>Window width</i> and <i>Window center</i>	34
6.5.2	<i>ROI opacity</i>	35
6.5.3	<i>Orientation labels</i>	35
6.5.4	<i>3D Localizer</i>	35
6.5.5	Reset settings	35
6.6	Display the report	35
6.6.1	Navigating the report	36
6.6.2	Download the report	36
7	Troubleshooting	37
7.1	I can't access the visualization interface	37
7.2	I can't send data from my DICOM server	37
7.3	I can't send data from the visualization interface	38
7.4	I do not receive the results of my analysis after the indicated time	38
7.5	I can't find the exam that I wish to visualize	38
7.6	I can't access the account management page	38
	Index	39

CHAPTER 1

Legal notices

QYNAPSE		QyScore
	2-10 rue d'Oradour-sur-Glane 75015 PARIS FRANCE E-mail: support@qynapse.com Web: https://www.qynapse.com	 3 770016 198004  Manufactured in: 2-10 rue d'Oradour-sur-Glane 75015 PARIS FRANCE E-mail: support@qynapse.com

Caution: U.S. Federal law restricts this device to sale by or on the order of a physician.

1.1 Copyright notice and trademark acknowledgements

Copyright © 2025 by QYNAPSE SAS, all rights reserved.

No part of this manual may be copied, reproduced, modified, published or distributed in any form or by any means, for any purpose, without prior written permission of QYNAPSE SAS.

QyScore © is a registered trademark of QYNAPSE.

DICOM is the registered trademark of the National Electrical Manufacturers Association for its standards publications relating to digital communications of medical information. All other brand, product, and company names mentioned in this document are trademarks or registered trademarks of their respective owners.

1.2 Indications for use

QyScore is intended for automatic labeling, visualization and volumetric quantification of segmentable brain structures and lesions from a set of MR images. Volumetric data may be compared to reference percentile data. QyScore is not intended for use in clinical scenarios that require evaluation of the number of the white matter hyperintensities.

1.3 Contraindications

- Data with gross imaging artifacts and/or motion should not be sent for analysis as they could cause failure or incorrect segmentation of brain tissues.
- QyScore should not be used in case of gross anatomical abnormalities, resections or tumors, as these conditions could cause failure or incorrect segmentation of brain tissues.
- Corrupted data sent from the DICOM server won't be analyzed. You will be informed in that case.
- QyScore is not intended for use in clinical scenarios that require evaluation of the number of the WMH.

1.4 Target patient population

QyScore provides reliable and reproducible segmentations and measures of brain structures for patients between 20 and 90 years old. There are no specific diseases. QyScore can analyse any structural MRI of the brain compatible with the recommended protocol.

1.5 Intended users

The software is intended to be used by medical personnel or neuroimaging trained personnel to support diagnosis of central nervous system diseases.

These users are radiologists, neuroradiologists, neurologists, geriatricians, psychiatrists.

These users need to be able to visualize information displayed with the color codes in a dark or light room with multiple screen environment. Before using the software, the user must have completed a initial training program. This program ensures the training of other users.

1.6 Description

QyScore is intended for automatic labeling, visualization and volumetric quantification of segmentable brain structures and lesions from a set of MR images for patients between 20 and 90 year old. Volumetric data may be compared to reference percentile data. QyScore is not intended for use in clinical scenarios that require evaluation of the number of the white matter hyperintensities.

The software:

- retrieves imaging data from a DICOM server or folder,
- sends these data to Qynapse's analysis server,
- enables the user to view the results (segmentations and numerical results) in a dedicated interface,
- enables the user to download a summary report of the results (segmentations and numerical results),
- enables the user to get the results in a structured manner into a DICOM SR that can be use into other informations systems.

The visualization graphical interface allows the user to:

- Send imaging data to the QYNAPSE analysis server,
- Browse the segmentations and the measures,
- Compare the results of segmented brain structures to a reference healthy population,
- View and download the summary report.

This interface is a web application that must be used with a web browser.

The software is designed to be user-friendly and intuitive. It is intended to be used by medical personnel or neuroimaging trained personnel to support diagnosis of central nervous system diseases.

QyScore integrates with leading RIS/PACS systems, and can be operated with any MRI scan from 1.5T and 3T scanners for T1 MRI processing, and 3T scanners for T2FLAIR MRI processing.

Caution: U.S. Federal law restricts this device to sale by or on the order of a physician.

1.7 Clinical Benefits

When used as intended and in accordance with this user manual, no direct or immediate clinical benefit is claimed for QyScore. The software does not provide diagnostic or therapeutic recommendations. The clinical benefit is indirect: QyScore provides accurate, standardised, and quantitative MRI information that supports clinical decision-making by qualified healthcare professionals. The final patient outcome depends on further diagnostic and/or therapeutic steps.

Operational benefits for users include:

- Accuracy of quantitative measures comparable to expert manual methods
- Reproducibility consistent with or superior to state-of-the-art automated tools
- Objective and reliable longitudinal quantification for follow-up assessments
- Efficiency and standardisation gains in reporting workflows

Final clinical decisions remain under the sole responsibility of the clinician.

1.8 Global presentation

To start using QyScore, two modes of operation are available:

- Sending data directly from the DICOM server (PACS) (direct connection - see chapter 4.2),
- Using the web-based interface from your computer (see chapters 4.3 and 6)

1.9 Technical description

1.9.1 Installation

QyScore

QyScore is installed by the QYNAPSE team, on a computer with the following minimal requirements:

For less than 100 analyses/month:

Greater or equal to:

- RAM: 12 GB
- CPU: architecture x86_64, 6 vCPU minimum
- 500 GB disk capacity

For more than 100 analyses/month:

Greater or equal to:

- RAM: 24 GB
- CPU: architecture x86_64, 12 vCPU minimum
- 3 TB disk capacity

To ensure proper function of the software, this operating system must not be modified.

For the direct connection mode, the computer must be connected to a DICOM server to enable data sending to the analysis server.

Several configurations for the installation of QyScore are possible:

- Distant server: QyScore is located on a cloud server, with the data being stored outside the client organization, on a personal health data hosting system
- Local server: it is located on a computer of the client organization

Web-based user interface

The minimal requirements for the computer running the web-based user interface are:

- RAM: 4Go
- CPU: 4
- Screen size 1024*768 and resolution 132 PPI
- Mouse
- Keyboard

Several browser can be used:

- Mozilla Firefox, version 68.0.2 (64-bit) or higher
- Google Chrome, version 76.0.3809.100 (64-bit) or higher

If QyScore is installed remotely, the computer running the interface must be connected to the Internet to enable communication between client and server sides.

Caution: The computer running the interface must be equipped with appropriate firewall and antivirus systems. If they are not provided by QYNAPSE, it is the customer's responsibility to ensure that these items are installed.

1.9.2 Compatibility with other devices

QyScore retrieves DICOM MRI data from a DICOM server or the computer and delivers DICOM data (images and statistics).

1.9.3 Performances

QyScore automatically provides segmentations and measures of brain structures and lesions from a set of MR images. Features include the computation of:

- Total brain volume (Grey and white matter)
- Hippocampal volume
- Amygdala volume
- White matter hyperintensities load
- Evolution of white matter hyperintensities load over time

The performances of QyScore were evaluated by comparing the segmentation accuracy with expert manual segmentations, using relevant metrics and thresholds that are widely used and accepted in the literature.

In the conditions detailed in section 1.6, the performances of the quantification methods on 3DT1 scans are the following:

Structure	# Sub- jects	Age Range	Sex (M-F)	Field strength (1.5T-3T)	Dice (mean ± std)	Relative volume error (% mean ± std)
Grey matter	22	20 - 90	9 - 13	11 - 11	0.85 ± 0.03	9.07 ± 5.46
White matter	22	20 - 90	9 - 13	11 - 11	0.90 ± 0.02	7.09 ± 5.61
Hip- pocam- pus	34	20 - 90	16 - 18	19 - 15	0.81 ± 0.04	21.69 ± 7.85
Amyg- dala	34	20 - 90	16 - 18	19 - 15	0.78 ± 0.05	14.24 ± 16.06

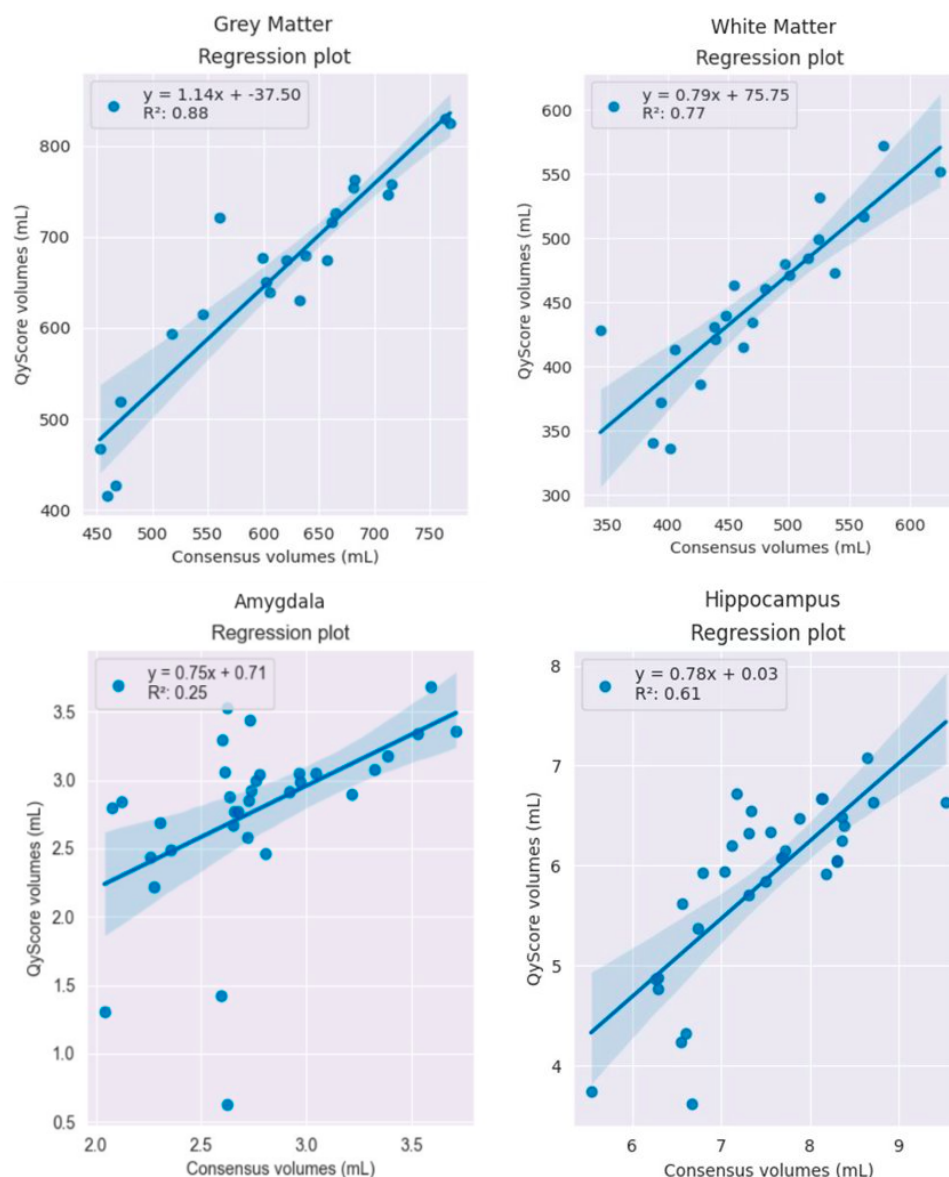


Fig. 1.1: Validation regression plots of grey matter, white matter, hippocampus and amygdala.

Note: QyScore tends to always overestimate grey matter volume. QyScore tends to underestimate white matter, especially when the volume is high.

Note: QyScore always underestimates the hippocampal volume.

The white matter lesion load is computed on T2FLAIR scans, paired with the corresponding 3DT1 scans. Only 3T scans are accepted for this measure. The performances were assessed per lesion load subgroups. Results are:

Sub-group	# Sub-jects	Age Range	Sex (M-F)	Type (2D-3D)	Dice (mean $\pm$ std)	Absolute volume error (mL, mean $\pm$ std)
< 5 mL	13	26 - 74	3 - 10	4 - 9	0.40 $\pm$ 0.16	0.38 $\pm$ 0.3
5 - 15 mL	12	29 - 57	6 - 6	5 - 7	0.66 $\pm$ 0.1	2.1 $\pm$ 1.63
15 - 30 mL	14	30 - 80	4 - 10	7 - 7	0.75 $\pm$ 0.09	3.35 $\pm$ 3.11
> 30 mL	12	25 - 88	3 - 9	0 - 12	0.78 $\pm$ 0.09	9.04 $\pm$ 5.22

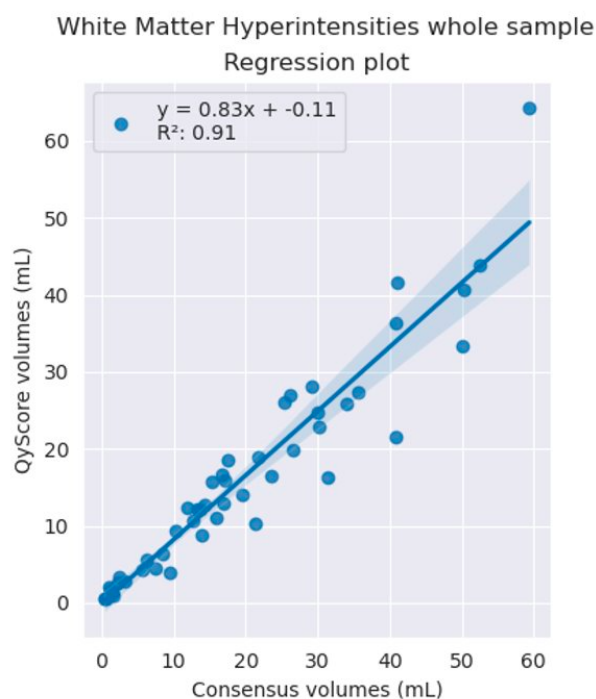


Fig. 1.2: Validation regression plots of white matter hyperintensities.

In addition, for white matter lesion load quantification, the lesion-wise true and false positive rates are respectively 0.43 ± 0.21 and 0.46 ± 0.27 .

Note: QyScore is not intended for use in clinical scenarios that require evaluation of the number of the WMH.

The results of the increased/decreased/static volumes of lesions obtained on either sequential VS or direct methods are displayed in the following table with associated correlation and R2 coefficient :

Structure	# Sub-jects	Age Range	Sex (M-F)	Type (2D-3D)	Field strength (1.5T-3T)	R ² in-creased	R ² de-creased	R ² static
WMH over time	6	62 - 84	0 - 6	6 - 0	1 - 5	0.92	0.87	0.90

The volumes provided by the software are indicated as absolute and relative (in % of intracranial volume) values.

1.9.4 Maintenance

The computer containing the analysis server must be turned on for the software to function properly. In case of a planned power cut, it shall therefore be turned back on. The database with the analysis results is automatically purged when the maximum storage capacity is reached.

This device is designed for an operating lifetime of up to 3 years, during which Qynapse provides maintenance and support to ensure optimal performance and compliance. The exact duration is defined as the shorter period between installation date + 3 years and the timeframe specified in the signed Commercial Contract, whichever applies.

1.9.5 Deactivation, Uninstallation and Data Disposal

Deactivation/uninstallation of QyScore is performed by QYNAPSE upon request to support@qynapse.com.

For deactivation, QYNAPSE disables the access to the viewer and the mechanism that sends results back to PACS are disabled.

For uninstallation, QyScore is uninstalled and user data are deleted.

The customer remains responsible for any data stored outside the device, such as PACS or institutional archives/backups. Upon completion, the customer receives a confirmation that the software has been deactivated or uninstalled and that user data deletion has been performed.

1.9.6 Safety

If a safety breach is detected, the analysis server will be shut down by Qynapse to avoid potential fraudulent use. In that case, new data analyses won't be possible, but the analyses previously performed will remain available on the DICOM server.

1.10 Required imaging parameters

QyScore is designed to analyze non-injected 3DT1 (1.5T or 3T), as well as T2FLAIR (1.5T or 3T). The required acquisition parameters are the following:

	T1	
	Required	Tolerated
Voxel size in plane (mm)	[1, 1]	Min: [0.43, 0.43] Max: [1.5, 1.5]
Field of view (mm)	x: [152, 340] y: [220, 340] z: [152, 340]	x: [144, 340] y: [170, 340] z: [132, 340]
Slice thickness (mm)	1	Min: 0.68 Max: 2
Gap between slices (mm)	0	Min : -1 Max : 0
Interpolation	No	Yes
Acquisition direction	Sagittal	Axial, Coronal
Acquisition type	3D	2D with isotropic voxel
Field strength	1.5T, 3T	1.5T, 3T

	T2FLAIR2D		T2FLAIR3D	
	Required	Tolerated	Required	Tolerated
Voxel size in plane (mm)	[1, 1]	Min: [0.43, 0.43] Max: [1.1, 1.1]	[1, 1]	Min: [0.43, 0.43] Max: [1.1, 1.1]
Field of view (mm)	x: [152, 340] y: [220, 340] z: [152, 340]	x: [144, 500] y: [170, 340] z: [132, 340]	x: [152, 340] y: [220, 340] z: [152, 340]	x: [144, 340] y: [170, 340] z: [132, 340]
Slice thickness (mm)	3.3	Min: 2 Max: 5	1	Min: 0.8 Max: 2.0
Gap between slices (mm)	0	Min : 0 Max: 1	0	Min: -1 Max : 0
Interpolation	No	Yes	No	Yes
Acquisition direction	Axial	Coronal, Sagittal	Sagittal	Axial, Coronal
Acquisition type	2D	2D	3D	3D
Field strength	1.5T, 3T	1.5T, 3T	1.5T, 3T	1.5T, 3T

Caution: In case of non-compliance with the parameters given above, the obtained results won't be optimal, in terms of processing time as well as reliability. If the acquisition parameters are outside of the tolerated range, the analysis won't be launched.

In addition, a quick review of the MR data to be sent for analysis is recommended to avoid sending data with obvious imaging artifacts, such as:

- brain not fully covered
- severe aliasing
- bad grey/white contrast
- severe head motion
- severe dental and/or other metallic artifact

These artifacts could cause failure or incorrect segmentation of brain tissues.

1.11 Normative data

The statistics generated by QyScore (Z-scores and percentiles) are computed by comparison to results obtained on a normative database of healthy subjects.

The database is a concatenation of the following databases:

Database	Mean age	Std age	Minimum age	Maximum age	Healthy subjects	Females	Males	% males
1	59.5	14.7	20	89	805	436	369	46
2	67.1	9.5	43	97	720	397	323	45
3	48.7	16.5	20	86	563	313	250	44
4	38.5	18.3	18	84	468	238	230	49
5	52.2	23.5	20	94	306	191	115	38
6	27.5	4.8	23	38	172	87	85	49
7	34	12.8	18	72	155	102	53	34
8	31.3	8.7	21	50	122	57	65	53
9	78.2	6.8	60	93	57	35	22	39
10	67.1	8.5	46	85	47	26	21	45

1.12 Warnings

Users must make sure to read this manual to become familiar with the software and be able to obtain reliable analysis results.



QyScore is intended to be used by medical personnel or neuroimaging trained personnel. The software should not be used for purposes other than those indicated in the section 1.2.



Staying in front of a computer for too long may make your muscles, joints or eyes hurt. If your muscles, joints or eyes become tired or sore, take a break in the use of the software.



QyScore does not include accessibility modes.



The fully automated segmentations computed by QyScore can be negatively impacted by image quality (such as motion artefact) and significant neuromorphological abnormalities (such as emerging tumors) as highlighted in section 1.3 above. Therefore, in the presence of such issues, there may be some mis-segmentation and it is recommended the expert clinician visually assess the accuracy of these segmentation overlays to confirm the accuracy of the volume outputs reported.



Considering the lack of measure precision for the input DICOM data, QYNAPSE cannot guarantee the precision of volumes that are expressed in mL. Said volumes are given as indications only. Please refer to relative volumes (in % of intracranial volume) for reliable results.



The patients for whom QyScore analyzes the data must be between 20 years old and 90 years old.



QYNAPSE is not responsible for the management and maintenance of third-parties tools such as the DICOM server, nor for the quality of the infrastructure (except for the offered material). If QyScore is installed locally, the customer is responsible for the control of physical access to the server.



The installation should be supervised by QYNAPSE personnel.

1.13 Reporting of Serious Incidents

If you experience or observe an event involving QyScore that has resulted in death, serious injury, or a malfunction that could potentially lead to harm, you are encouraged to report the event to Qynapse without delay.

Please contact: Qynapse Support, support@qynapse.com

Qynapse will evaluate the event and, if applicable, submit a report to the U.S. Food and Drug Administration (FDA) in accordance with the Medical Device Reporting (MDR) requirements under 21 CFR Part 803.

1.14 Electronic instructions for use

Please note that QyScore software is supplied with electronic instructions for use. The  icon signals that additional information or essential instructions are available in the user manual.

These instructions may be accessed in the following ways:

- **Through the web-based user interface, by clicking the  icon**
 - by clicking on **About** and then on the  icon.
 - by clicking **User Manual**.
- From the online Qynapse document library: <https://manuals.qynapse.com>.

Please note that the instructions for use on PC are provided in **PDF** format. A suitable reader, such as Adobe Acrobat Reader is required to view them. Adobe Reader may be obtained from <https://get.adobe.com/reader>.

A hard-copy of the instructions for use may be obtained within 7 calendar days at no additional charge by contacting Qynapse at the following address: 2-10 rue d'Oradour-sur-Glane, 75015 Paris, France.

The following conventions are used throughout this manual.

2.1 Mouse

Click

Press and release the primary mouse button. If you are left-handed, you may have set the right mouse button as your primary mouse button.

Double-click

Press and release the primary mouse button twice. If you are left-handed, you may have set the right mouse button as your primary mouse button.

Long click

Press the primary mouse button and move the mouse. If you are left-handed, you may have set the right mouse button as your primary mouse button.

Scroll

Roll the wheel button.

Right-click

Press and release the secondary mouse button. If you are left-handed, you may have set the left mouse button as your secondary mouse button.

Middle-click

Press and release the wheel button or the middle mouse button.

Long middle-click

Press the wheel button or the middle mouse button and move the mouse.

2.2 Typography

OK

Buttons, fields and clickable menus names in the user interface are capitalized and in bold.

Login

Non-clickable labels in the user interface are indicated in italic.

Settings > Configuration

A sequence of menu options that you select to perform a specific task is indicated by angular brackets and in bold.

2.3 Symbols



Warns you to pay extra care to a specific point while using QyScore.

2.4 Glossary

Standard visualization window (or standard window)

In the visualization interface, simple volume visualization window.

False colors

Color applied to a zone or an object in a grey-scaled image, with the only purpose to highlight it. This color is chosen arbitrarily and any resemblance with the actual color of the zone or object is purely coincidental.

Special window

In the visualization interface, visualization window which content depends on the view mode.

Z-score

Standard deviation to mean value.

Percentile

Value that divides the population in 100 equally-sized groups (each percentile corresponds to 1 % of the population).

3.1 Global presentation

To connect to the visualization interface from your computer, you need to open a web browser and enter the URL provided by QYNAPSE.

3.2 Login

Enter your username and password in the corresponding fields. The password is initially given by your administrator or by QYNAPSE upon installation.

Click on **Sign in**.

If the username and password are correct, the interface is launched.

3.2.1 Case of first login

If you log in for the first time, you'll have to use the forget password procedure.

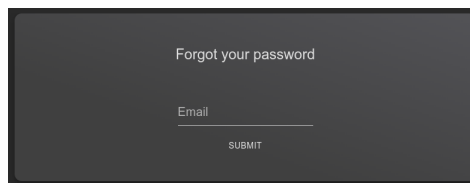


Fig. 3.1: Forgot your password

To do so:

- *Email*: type your email
- Click on **Submit**

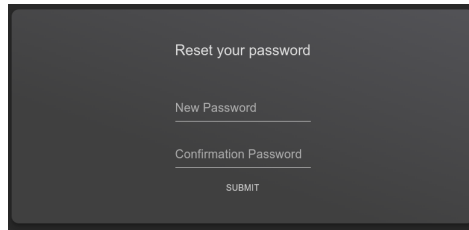


Fig. 3.2: Reset your password

To do so:

- *New password*: type your new password
- *Confirm new password*: confirm your new password
- Click on **Submit**

Note: This new password will be required for your next login.

3.3 Change password

Follow the steps:

- Click on the  icon
- Click on **Change password**
- *Current password*: type your current password
- *New password*: type your new password
- *Confirm new password*: confirm your new password
- Click on **Change password** to save the new password
- Click on **Cancel** to cancel the procedure and close the window

3.4 Logout

To logout:

- Click on the  icon
- Click on **Sign out**

3.5 Forgotten password

If you forget your password, please contact QYNAPSE at support@qynapse.com.

4.1 Global presentation

Data can be sent to QYNAPSE's analysis server:

- directly via your DICOM server (e.g. PACS)
- via the web-based data management interface

In all cases, the received data are automatically analyzed.

Caution: The analysis that will be performed depends on the type of sent data and on the license that you subscribed with QYNAPSE.

Caution: The processing time depends on the sequences' modalities and on the images resolutions. For your information, when following the recommendations, the processing times are 10min and 20min for a 3DT1 sequence and for a (3DT1, T2FLAIR) group respectively.

4.2 Via the DICOM server (e.g. PACS)

If properly configured, you can send data directly from your DICOM server, by selecting the sequences to analyze and exporting them to QyScore. Please refer to the user manual of your DICOM server for further information.

You can also export the whole exam for a given patient, but the processing time will be longer.

The export of all the exams for a given subject is currently not handled.

Once the analysis is finished, the results are sent directly in DICOM format to your server.

You can view them using the visualization interfaces. For further information and description of features, see chapter 6.

Caution: The database is regularly cleaned to save disk space. As a consequence, if the exam you wish to view is no longer available, please relaunch the analysis.

4.3 Web-based data management interface

4.3.1 Uploading data

Data can be sent using the *Launch new analyses* interface. To open this interface:

- Click on the  icon

Or:

- Click on the  icon
- Click on **Launch new analyses**

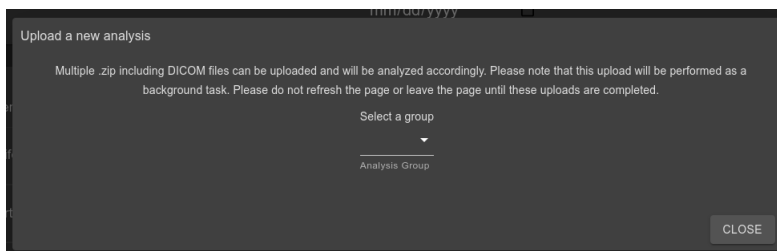


Fig. 4.1: Launch new analyses

In the *Launch new analyses* interface:

- Drag and drop the files in the upload zone or click on the upload zone to browse
- Wait for the uploading to finish. The *Ready* message should be displayed.

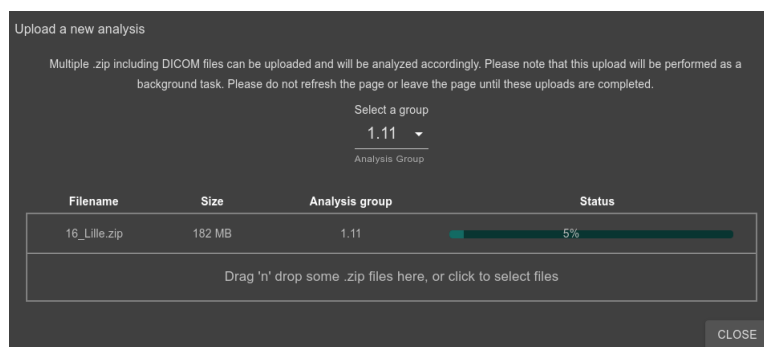


Fig. 4.2: Launch new analyses - Uploaded file

Note: You must compress your DICOM folders (.zip format) before uploading them via the interface.

Note: The .zip file can contain DICOM data for several patients/acquisitions. They will be automatically sorted and the analysis will be launched for each patient and acquisition.

Remove an uploaded file

To remove a file from the upload zone, click on the  icon.

Send data

Once the files are uploaded, click on **Send**.

A confirmation pop-up is displayed.

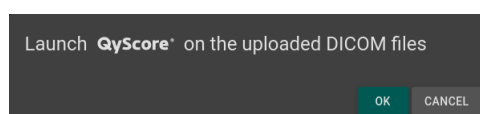


Fig. 4.3: Confirmation for sending data for analyses


- Click on **Ok** to send the uploaded data for analyses
- Click on **Cancel** to cancel the procedure and close the window

The chosen data are copied and sent to QYNAPSE's analysis server. You can follow the progress and monitor the launched analyses in the *Analyses monitoring* interface.

4.3.2 Analyses monitoring

Launched analyses can be monitored in the *Analyses monitoring* interface. To open this interface:

- Click on the  icon
- Click on **Analyses monitoring**

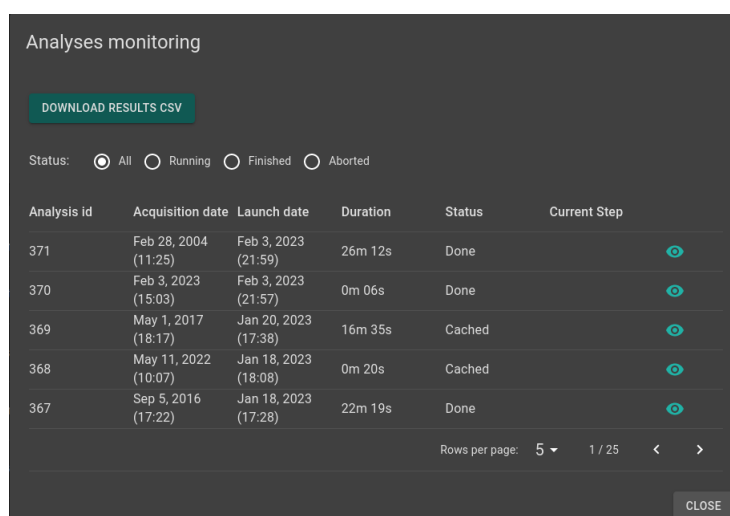A screenshot of the "Analyses monitoring" interface. It features a dark gray background. At the top, there's a "DOWNLOAD RESULTS CSV" button. Below it, a "Status:" label is followed by four radio buttons: "All" (selected), "Running", "Finished", and "Aborted". A table with six columns follows: "Analysis id", "Acquisition date", "Launch date", "Duration", "Status", and "Current Step". The table contains five rows of data. At the bottom right, there's a "CLOSE" button. The table also includes pagination controls at the bottom right showing "Rows per page: 5" and "1 / 25".

Analysis id	Acquisition date	Launch date	Duration	Status	Current Step
371	Feb 28, 2004 (11:25)	Feb 3, 2023 (21:59)	26m 12s	Done	
370	Feb 3, 2023 (15:03)	Feb 3, 2023 (21:57)	0m 06s	Done	
369	May 1, 2017 (18:17)	Jan 20, 2023 (17:38)	16m 35s	Cached	
368	May 11, 2022 (10:07)	Jan 18, 2023 (18:08)	0m 20s	Cached	
367	Sep 5, 2016 (17:22)	Jan 18, 2023 (17:28)	22m 19s	Done	

Fig. 4.4: Analyses monitoring

The table displays information about the analyses:

- *Patient*: the patient's name or identifier
- *Acquisition date*: the last study date of the patient's exams
- *Launch date*: the launch date of the analysis
- *Duration*: the duration of the analysis
- *Status*: the status of the analysis. It can be:
 - *Pending*: the analysis is waiting to be processed (process is in queue)
 - *Scheduled*: the analysis is scheduled for execution (process is about to start)
 - *Running*: the analysis is running (a progress bar is displayed)
 - *Done*: the analysis is completed successfully
 - *Error*: the analysis encountered an unexpected error
 - *Aborted*: the analysis was cancelled by a user
 - *Cached*: the analysis is completed successfully but some steps were skipped because already computed in a previous analysis
- *Current Step*: the analysis evolution in real time

Cancel a *Scheduled* analysis

Scheduled analyses can be cancelled before they start to run.

To cancel a *Scheduled* analysis:

- Click on the  icon of the corresponding analysis row

A confirmation pop-up is displayed.

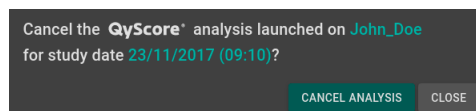


Fig. 4.5: Confirmation for analysis cancellation

- Click on **Cancel analysis** to cancel the *Scheduled* analysis. It sets its status to *Aborted*.
- Click on **Close** to cancel the procedure and close the window

Note: If the analysis changes from *Scheduled* to *Running* while the confirmation pop-up is opened, the analysis is not cancelled and the message *Your analysis cannot be cancelled, it has already started* is displayed.

Filter analyses

To filter the analyses according to their status:

- Click on one of the following:
 - *All*: displays all the analyses
 - *Running*: displays *Running* analyses
 - *Finished*: displays *Finished* analyses
 - *Aborted*: displays *Aborted* analyses

Load the results of a *Finished* analysis

Results of *Finished* analyses can be loaded for visualization. To do so:

- Click on the  icon of the analysis row, for the selected patient/acquisition

A confirmation pop-up is displayed.

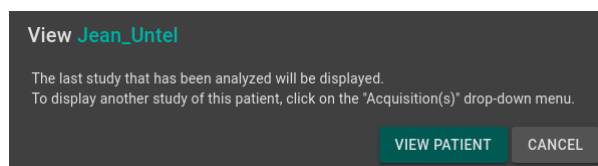


Fig. 4.6: Confirmation for loading a *Finished* analysis

- Click on **View patient** to load the patient onto the interface
- Click on **Cancel** to cancel the procedure and close the window

Note: Once the patient is loaded onto the interface, the last study that has been analyzed is displayed.

To display another study of the loaded patient, click on the  icon of the *Acquisition(s)* drop-down menu and select the corresponding study date.

Download the results

To obtain a file with the results of completed analyses:

- Click on **Download CSV results**

The results are presented in a CSV file, each line referring to a patient/acquisition, and each column referring to a MRI marker.

5.1 Global presentation

Three templates of reports are available :

- *Generic*: with all QyScore brain quantifications and graphs.
- *Dementia*: with QyScore brain quantifications and graphs specific to follow-up of patients with dementia.
- *Multiple Sclerosis*: with QyScore brain quantifications and graphs specific to follow-up of patients with multiple sclerosis.

Each report contains a summary of QyScore analysis results.

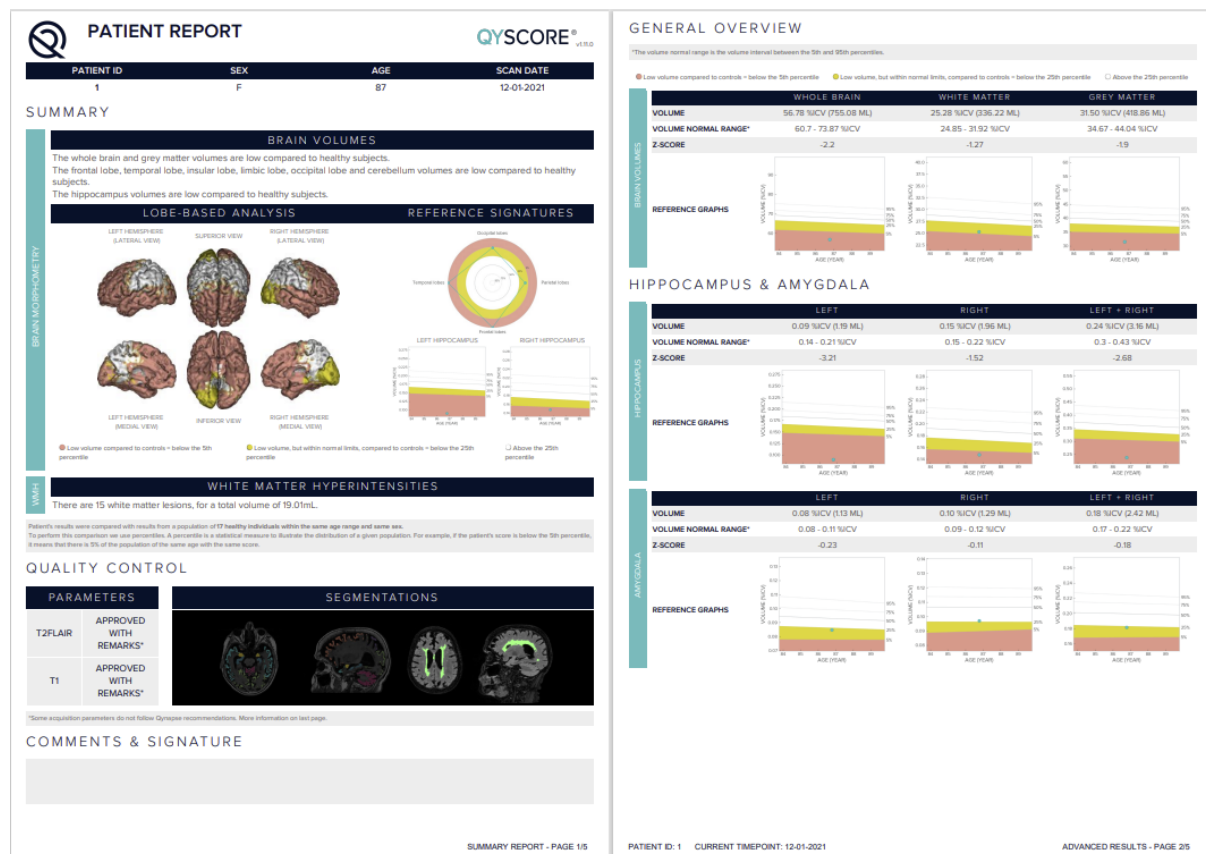


Fig. 5.1: Report

The sections that are available for all reports are the following ones:

- *Patient information*: patient-related information (name, age, sex), acquisition date.
- *Summary*: summary of results obtained on the segmented structures.
- *Overview*: detailed automated results, including numerical results and graph of comparison to normative data (described in chapter 1.11).
- *Quality control*: detailed results of the automated quality control performed by QyScore.
- *Comments & Signature*: free space to edit comments and to sign.

The section that is specific to a patient with 2 timepoints is the following one:

- *Longitudinal analysis*: longitudinal results computed automatically on anatomical structures and brain lesions.

Note: The *Longitudinal analysis* field is not visible if only one timepoint is available for the selected subject.

The section that is specific to the Dementia report is the following one:

- *Lobe-based analysis*: contains images resulting from the lobes analysis of the 3DT1 MRI, shown in different views.

5.2 Display the report

The report can be displayed and printed using the web-based visualization interface (see [6.6](#)).

Web-based results visualization interface

6.1 Global presentation

This web-based visualization interface is designed to view the results computed by the analysis server. It contains three pages:

- A login page
- A general page for:
 - The management of analyses:
 - * Launch new analyses
 - * Monitor launched analyses
 - The visualization of the results:
 - * Interacting tools to navigate in the MR volumes, segmented structures and corresponding statistics
 - * Tools to view and download the summary report that contains the analysis results
- An account management page for users administration

6.1.1 Login page

You can connect to the interface through the login page. See [3](#) for more information.

6.1.2 Visualization page

Once logged onto the interface, the visualization page is displayed.

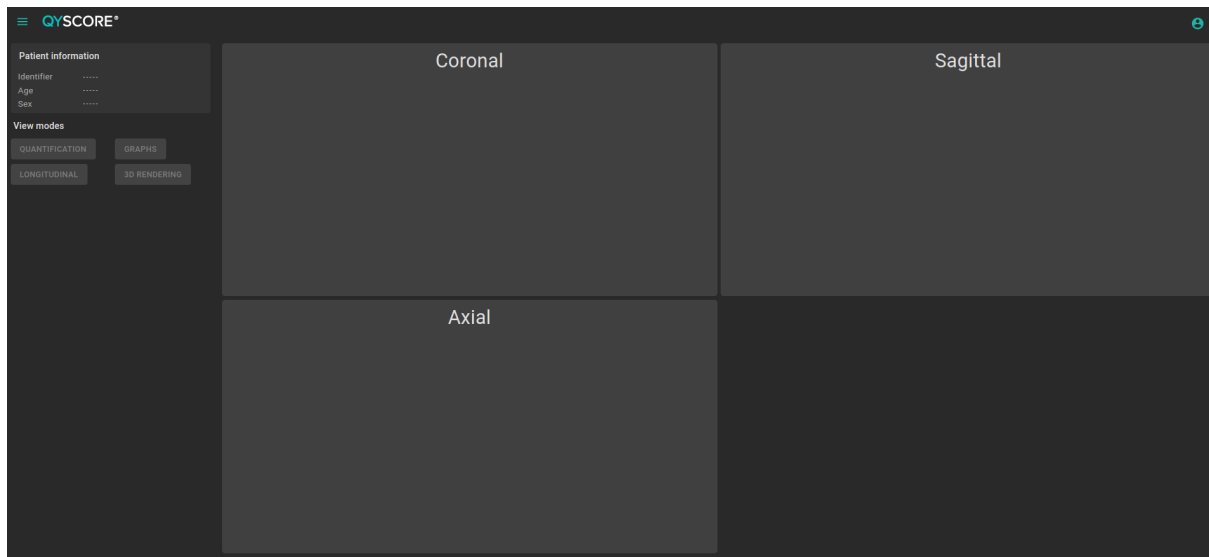


Fig. 6.1: Web-based interface visualization page

After loading patient data (as described in chapter 6.2):

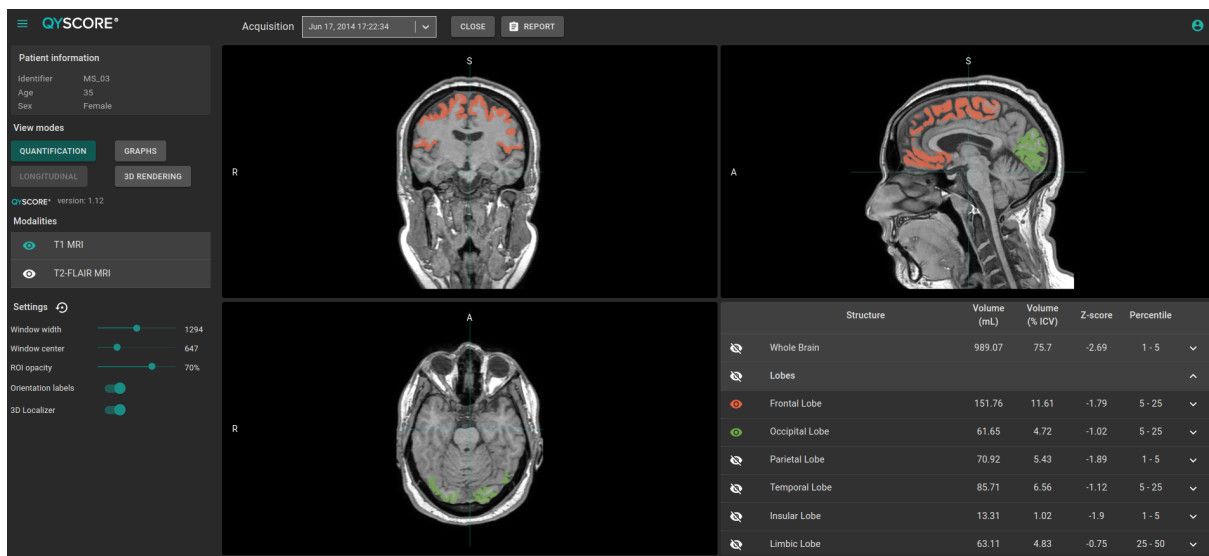


Fig. 6.2: Web-based interface visualization page - Patient loaded

The top bar is divided into several sub-parts:

- Side menu: accessible by clicking on the  icon, it provides access to different features:
 - In the *Analyses* section:
 - * **Launch new analyses** enables to start a new analysis by loading DICOM files
 - * **Analyses monitoring** opens the analysis monitoring tool
 - In the *MRI visualization* section:
 - * **Viewer** allows you to go back to the visualization page

- * **Load patient files** allows you to load analyzed patient data onto the interface
 - **User manual** displays QyScore user manual
 - **Shortcuts** displays keyboard and mouse shortcuts
 - **About** displays information about QyScore
-
- *Acquisition(s)* menu: to choose the study exam to display for the selected patient
 - *Report* menu: accessible by clicking on the **Report** icon, it allows you to display and download the summary of analysis results
 - *Launch analysis* window: accessible by clicking on the  icon, it allows you to launch a new analysis by loading DICOM files
 - *Load patient files* window: accessible by clicking on the  icon, it allows you to load analyzed patient data onto the interface
 - *Notifications* menu: accessible by clicking on the  icon, it allows you to consult the list of analyses for which feedback has not been provided
 - *My account* menu: accessible by clicking on the  icon, it allows you to manage users' accounts

The control panel on the left-hand side is divided into several sub-parts:

- *Patient information*: information on the selected patient
- *View modes*: to switch between view modes (default is *Quantification*)
- *Modalities*: to choose the displayed imaging modality
- *Settings*: to modify visualization options for better visual confort

The viewing part on the right-hand side contains three standard visualization windows that display the volume in the standard orthogonal orientations (coronal, sagittal, axial).

The fourth window is called “special window” and its content depends on the chosen view mode.

6.1.3 Account management page

Administrators can access this page by clicking on the  icon, then on **Manage account**. They can create new users or delete existing users of their organization.

6.2 Select patient data

6.2.1 Choose a patient to load

Follow the steps:

- Click on the  icon
- Click on **Select a database...** to select the relevant database
- Click on **Select a center...** to select the relevant center
- Click on **Type a patient ID...** to filter the patients according to their IDs
- Click on a patient's name to select them
- Click on **Download** to load the selected patient onto the interface
- Click on **Cancel** to cancel the procedure and close the window

Note: It may be necessary to re-open the window if new analyses have been performed while it was opened.

6.2.2 Change the acquisition to display

Once a patient is loaded onto the interface, the last study that has been analyzed is displayed. To display another study of the selected patient, click on the  icon of the *Acquisition(s)* drop-down menu and select the corresponding study date.

6.3 General visualization

The following instructions can be used regardless of the chosen view mode. The control panel is located on the left-hand side of the visualization page.

6.3.1 Browsing the volumes

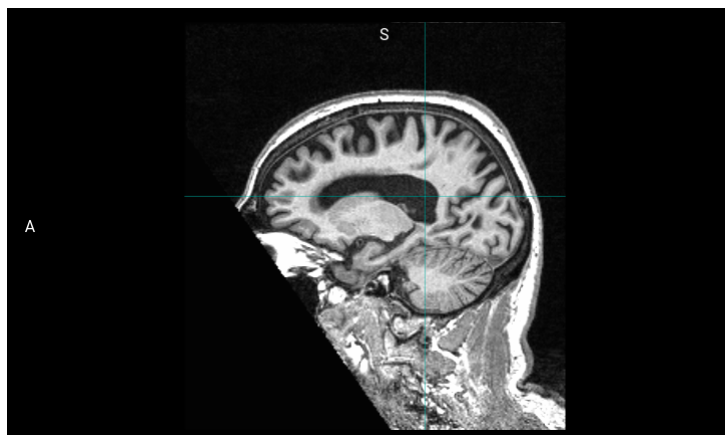


Fig. 6.3: Standard visualization window

The window contains:

- the volume,
- orientation labels: R for Right, S for Superior, A for Anterior,
- a 3D cursor that indicates the point on which the windows are adjusted.

Mouse controls

On the displayed volume:

- Click on any point of the volume: adjusts the other standard viewing windows to the same point. Also works with a long click.
- Double-click: set/unset the view to full size
- Scroll: changes the slice to display
- Long middle-click: moves the volume in the window
- Right-click: zoom in/out the volume. Also work with a long click.

6.3.2 View imaging modalities

Imaging modalities that can be displayed are indicated in the *Modalities* part of the control panel. These modalities were sent by the user and selected by QyScore for analysis.

To display or hide a modality:

- Click on the modality name or on the  /  icon on its left-hand side.
-  : modality displayed
-  : modality masked

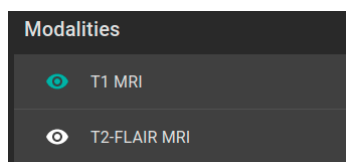


Fig. 6.4: Choice of modalities

6.3.3 View segmented structures

	Structure	Volume (mL)	Volume (% ICV)	Z-score	Percentile	
	Whole Brain	1188.57	86.15	0.57	50 - 75	▼
	Brain substructures	-	-	-	-	▲
	Amygdala	3.65	0.26	2.32	> 95	▲
	Left Amygdala	1.57	0.11	1.23	75 - 95	
	Right Amygdala	2.09	0.15	2.74	> 95	
	Hippocampus	7.18	0.52	2.25	> 95	▼

Fig. 6.5: Choice of segmented structures

The list of segmented structures that can be displayed is indicated in the special window. These structures were segmented by QyScore from the imaging data sent for the selected patient and exam.

They are grouped in the following way:

- Group of structures (e.g.: *Brain substructures*)
 - Whole structure (e.g.: *Amygdala*)
 - * Lateral structure (e.g.: *Left Amygdala*)

To display/hide a whole structure in the table:

- Click on the group name or the  /  icon on the right-hand side of the group of structures containing the whole structure.

To display/hide a lateral structure in the table (if applicable):

- Click on the structure name or the  /  icon on the right-hand side of the whole structure.

View the segmentations

Displayed structures can be viewed in “false colors” in the standard windows.

To display/hide a group of structures:

- Click on the  /  icon on its left-hand side: every structure of the group is either displayed or hidden.
-  : indicates that at least one structure of the group is displayed.
-  : indicates that no structure of the group is displayed.

To display/hide a whole structure:

- If it does not appear in the table: see previous section
- Click on the  /  icon on its left-hand side: the structure is either displayed or hidden (both sides, if applicable).
-  : indicates that at least one side of the structure is displayed.
-  : indicates that the structure is hidden.

To display/hide a lateral structure (if applicable):

- If it does not appear in the table: see previous section
- Click on the  /  icon on its left-hand side.
-  : indicates that the side is displayed.
-  : indicates that the side is hidden.

View the numerical results

Numerical results can be viewed in the special window when the *Quantification* mode is selected :

- the absolute volume in mL,
- the relative volume in % of intracranial volume,
- the z-score computed by comparison with normative data (described in chapter 1.11),
- the percentile computed by comparison with normative data (described in chapter 1.11).

Note: Z-scores and percentiles are computed only for some structures.

6.4 Switch view modes

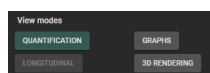


Fig. 6.6: View modes

Switching between view modes has an effect on the content of the special window and in some cases in the content of the standard window. Only one mode can be chosen at a time.

Available modes are described hereafter.

6.4.1 *Quantification* mode

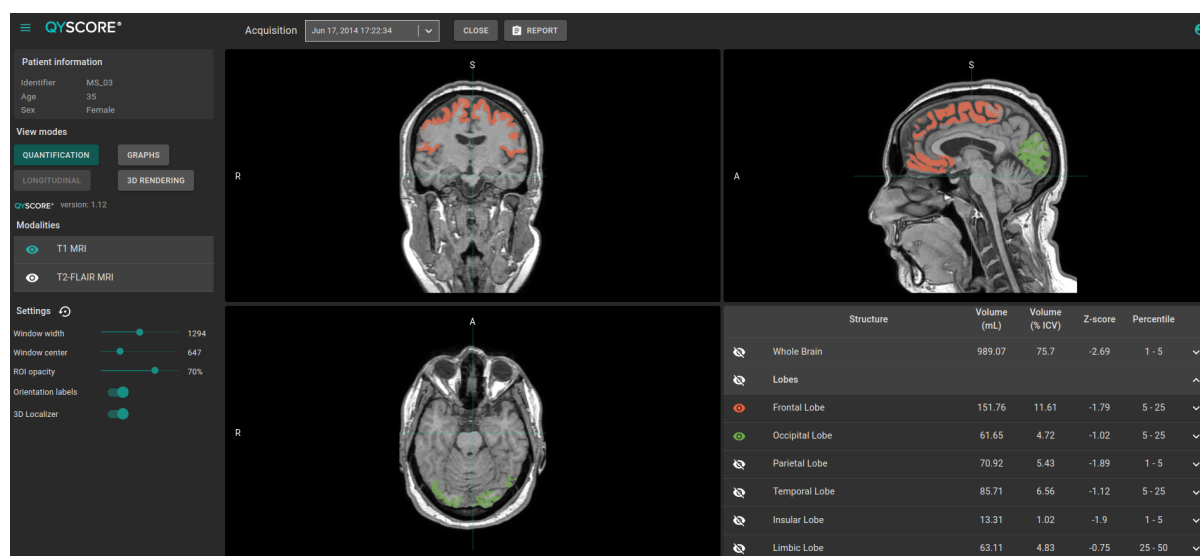


Fig. 6.7: *Quantification* mode

This is the default mode.

It displays the standard visualization of segmented structures and associated statistics, as described in chapter 6.3.

To enter the mode:

- Click on the **Quantification** button

The special window displays a table that contains:

- the list of the segmented structures that can be displayed,
- the numerical results related to these structures.

6.4.2 *Graphs mode*

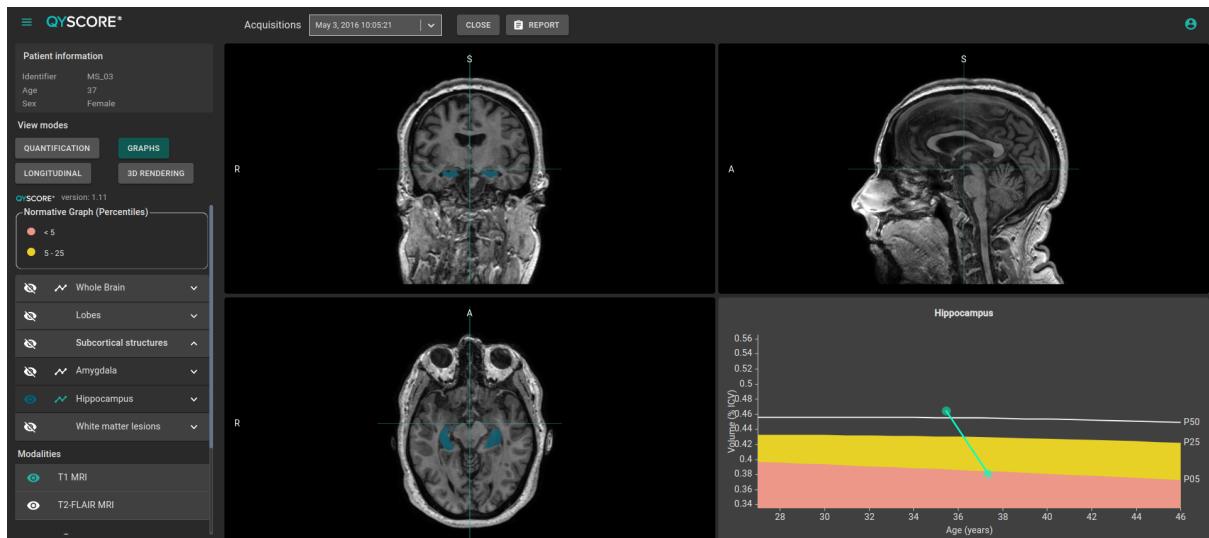


Fig. 6.8: *Graphs mode*

This mode enables you to view graphs of comparison with normative data (described in chapter 1.11).

To enter the mode:

- Click on the **Graphs** button

The special window displays a graph of comparison with normative data for the selected structure.

Note: Such graphs are only available for the structures in the *Whole brain* and *Brain substructures* groups.

Each graph displays the volume of the structure in comparison with the volumes computed on a normative dataset of subjects with similar age, described in chapter 1.11.

If several exams have been processed by QyScore for the selected patient, the volume of the structure at each timepoint is also displayed to enable a longitudinal follow-up.

Select a graph to display

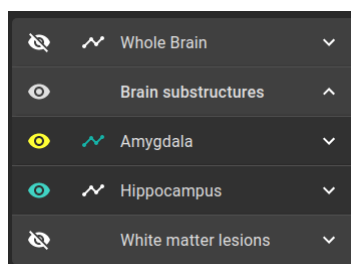


Fig. 6.9: Choice of displayed graph

The list of structures which graphs can be displayed is indicated in a table of the control panel. They are grouped in the same way as described in chapter 6.3.3.

To select the graph of a whole structure in the table (if applicable):

- Click on the  icon on the right-hand side of the whole structure.

To select the graph of a lateral structure in the table (if applicable):

- Click on the  icon on the right-hand side of the lateral structure.

6.4.3 Longitudinal mode

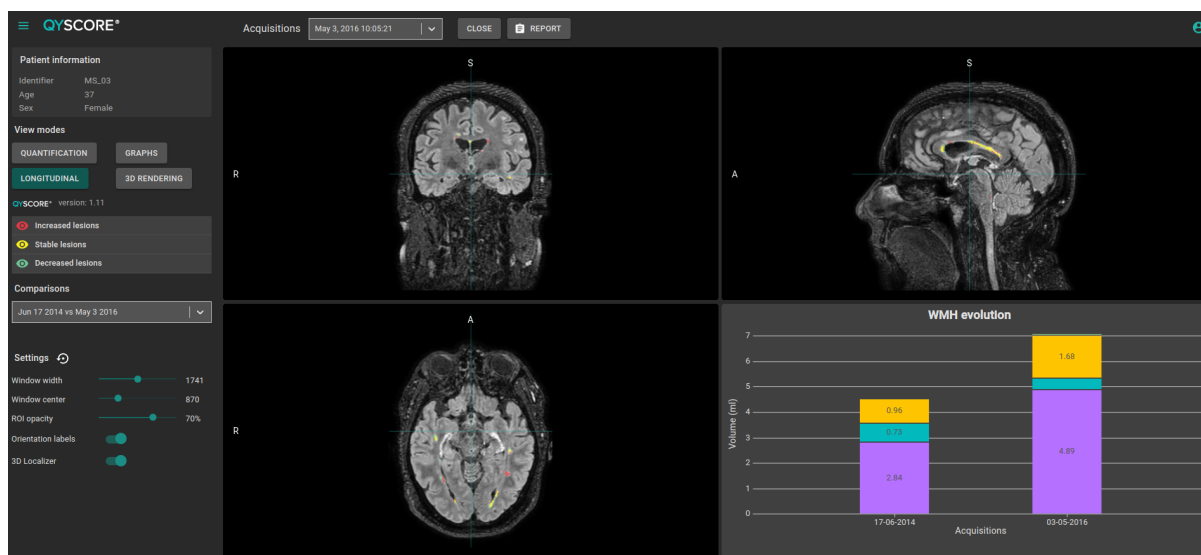


Fig. 6.10: Longitudinal mode

This mode enables you to view a graph of the lesion load evolution in time.

To enter the mode:

- Click on the **Longitudinal** button

This mode enables to visualize for a same patient the overlay of segmentations performed at several timepoints and to identify the increased, decreased and static lesions with the longitudinal scale.

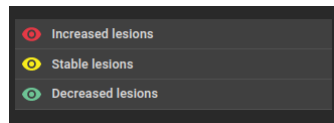


Fig. 6.11: *Longitudinal scale*

The special window displays a graph of the lesion load evolution in time.

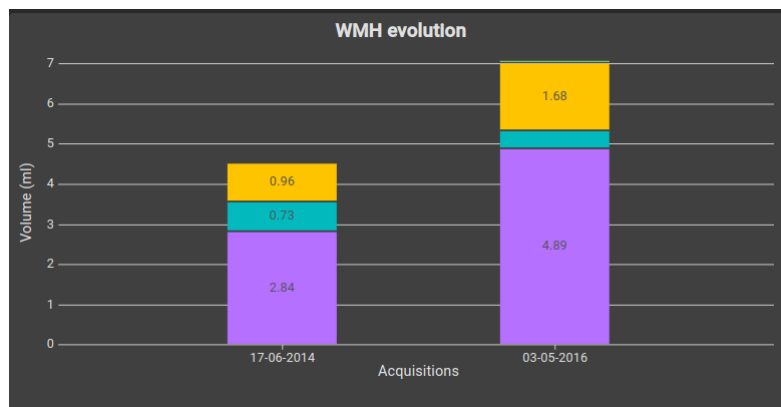


Fig. 6.12: *Longitudinal chart*

Note: To ensure accurate longitudinal analysis, it is recommended to use the same acquisition parameters at each time point.

6.4.4 3D rendering mode

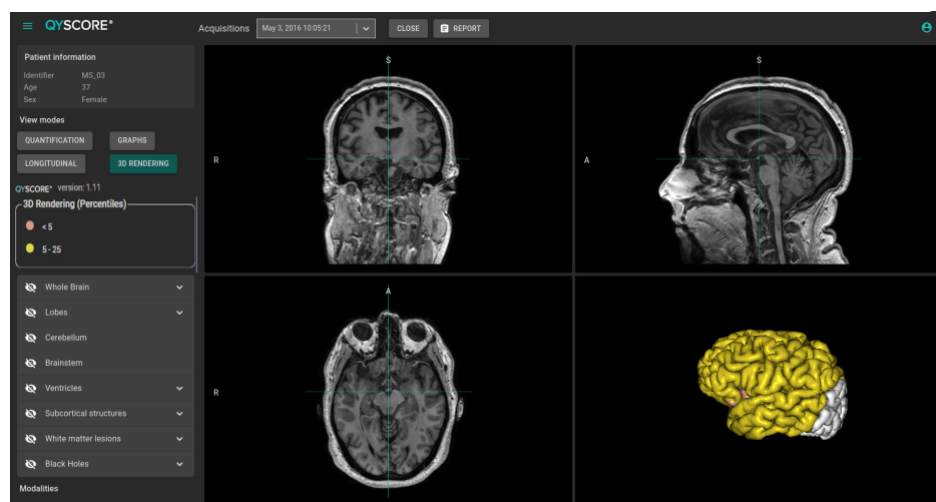


Fig. 6.13: *3D rendering mode*

This mode enables to visualize the atrophy in 3D and compare to the normative database according to the percentile scale.

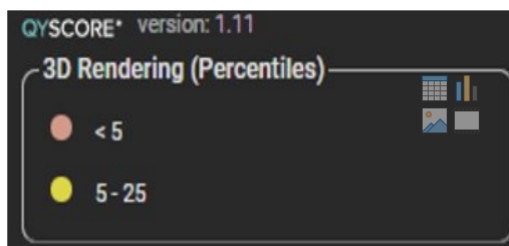


Fig. 6.14: 3D rendering scale

The 3D brain can be moved and zoomed with the mouse.

Note: The 3D brain rendering is displaying the projection of the patient percentiles on a standard 3D brain render.

Note: The 3D brain rendering is an anatomical view of the brain.

To enter the mode:

- Click on the **3D rendering** button

6.5 Change visualization settings

Visualization settings are available via the *Settings* section of the control panel.

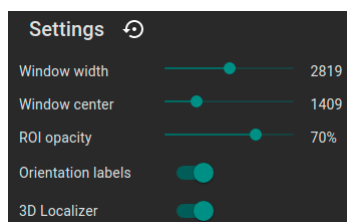


Fig. 6.15: Visualization settings

Changing the settings has a direct consequence on the displayed volume in standard windows.

Available settings are described hereafter.

6.5.1 Window width and Window center

These settings enable you to modify the intensity range of the displayed modality.

To change the intensity range:

- Long click on *Window width* or *Window center* sliders

6.5.2 ROI opacity

To change the ROI opacity:

- Long click on the *ROI opacity* cursor

Changing the cursor position changes the visibility of structures: the more it is positioned towards the right, the more they are visible.

6.5.3 Orientation labels

To enable/disable the display of orientation labels:

- Click on the *Orientation labels* switch button

6.5.4 3D Localizer

To enable/disable the 3D localizer:

- Click on the *3D Localizer* switch button

6.5.5 Reset settings

To restore default settings:

- Click on the  icon

6.6 Display the report

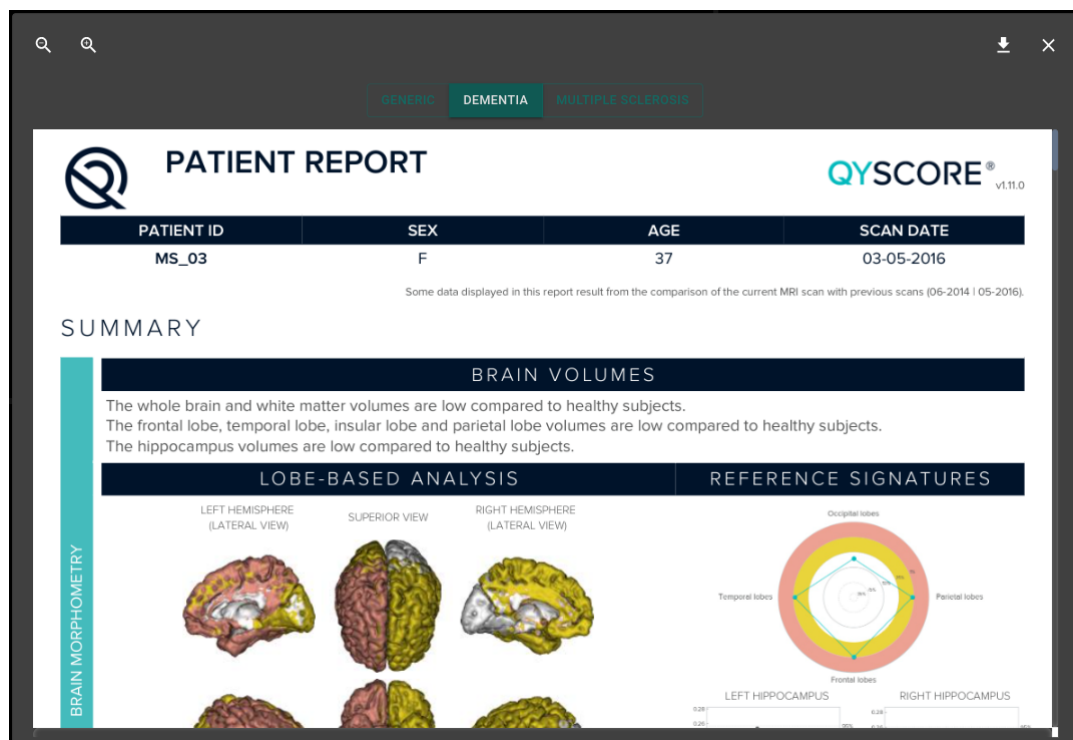


Fig. 6.16: Report visualization

To display the report:

- Click on the **Report** button
- Click on **View report**

Note: Volume changes are always computed by comparing with the first acquisition timepoint.

6.6.1 Navigating the report

To navigate:

- Browse through the pages: scroll
- Zoom in the report: click on the  icon
- Zoom out the report: click on the  icon
- Reset the size of the report: click on the  icon

Close the report

To close the report window, either:

- Click outside the report pop-up
- Click on the  icon

6.6.2 Download the report

To download the report, either:

- Click on the **Report** icon then click on **Download report**
- Click on the **Report** icon, then click on **View report**, then click on the  icon

The report will be saved in the “Downloads” folder of your computer.

Please refer to this section before requesting any further assistance.

7.1 I can't access the visualization interface

- Make sure that the URL you entered is correct.
- If Qynapse's server is installed locally, make sure that it is turned on.
- If Qynapse's server is installed remotely, make sure your computer is connected to the Internet.

7.2 I can't send data from my DICOM server

- If Qynapse's server is installed locally, make sure that it is turned on.
- Make sure that you have sent an exam or sequences.
- Make sure that you have chosen to send at least a 3DT1.
- Make sure that the chosen sequences are handled by QyScore.
- Make sure that the acquisition parameters are compliant with QYNAPSE's recommendations.
- Make sure that the configuration settings are correct (IP address, port...).
- If this does not solve the problem, please refer to the user manual of your DICOM server.

7.3 I can't send data from the visualization interface

- If Qynapse's server is installed locally, make sure that it is turned on.
- If Qynapse's server is installed remotely, make sure your computer is connected to the Internet.
- Make sure that you have uploaded a .zip (compressed format) of your DICOM files
- Make sure that you have chosen to send at least a 3DT1.
- Make sure that the chosen sequences are handled by QyScore.
- Make sure that the acquisition parameters are compliant with QYNAPSE's recommendations.

7.4 I do not receive the results of my analysis after the indicated time

In any cases:

- Wait a couple of minutes more: depending on the image resolution, the processing time can be longer than expected.

Via your DICOM server or the local data management interface:

- Try sending the data again, after making sure that there is at least a 3DT1, that those types of sequences are handled and that the acquisition parameters are compliant with Qynapse's recommendations.
- If this does not solve the problem, it may be due to a connection issue with the analysis server. The server is relaunched every minute in case of errors; wait a minute and try again.

Via the web-based interface, in the *Analyses monitoring* section:

- Wait a couple of minutes more: depending on the size of the uploaded files, the analyses can take more time to be displayed in the monitoring interface.

7.5 I can't find the exam that I wish to visualize

- Make sure that the filters applied in the searching tool are correct.
- In the web-based interface: if this analysis has been performed while the data loading window was opened, close and re-open the window.
- **Caution:** The database is regularly cleaned to save disk space. As a consequence, if the exam you wish to view is no longer available, please relaunch the analysis.

7.6 I can't access the account management page

- Make sure the session you are connected to is yours, by clicking on the  icon. Your username should be displayed on the top of the menu.
- If the message "You are not authorized to see this page" is displayed, it is because only administrators can access to this page.

F

False colors, [13](#)

P

Percentile, [13](#)

S

Special window, [13](#)

Standard visualization window (*or standard window*), [13](#)

Z

Z-score, [13](#)